

# Vivatmo pro

## How-to | Printing with Vivatmo pro

### Printing of patient reports with your Vivatmo pro

This instruction is intended to guide you through the installation process of your printer for printing a measurement report with the Vivatmo pro. In general, you have different options to connect a printer to the Vivatmo pro. The preferred printing methods are Network or Shared printing. These printing options are independent of the printer manufacturer. In addition, USB printing is available for most of the current HP printers and can be used if the other options are not possible.

### Setting up the printer configuration on your Vivatmo pro

#### INFORMATION

To set up a new or change an existing printer configuration you need to be logged in as an Administrator.  
You will find the Printer Configuration in the Administration Menu.

There are two ways to connect your printer to the Vivatmo pro basestation.

- Option A** Network and Shared Printing (recommended)
- Option B** USB Printing

### Option A: Network and Shared Printing (recommended)

#### INFORMATION


#### Preconditions

**Network Printing:** Your basestation must be connected to your network using WLAN or Ethernet. The network connection must be established before the printer can be configured. See administration instructions 5.2 Network configuration for detailed setup.

**Shared Printing:** Your basestation must be connected to your network using WLAN or Ethernet with an available print server (for further details contact your local IT responsible).


#### 1. Select Interface

- Select Network or Shared.

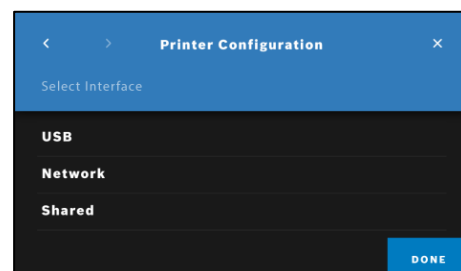

## 2. Select Printer


- The basestation will now search for available printers in your network.
- Choose your printer and press “DONE”.

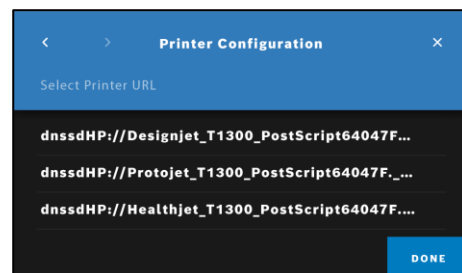

Printer name is an example

## 3. Select Driver


- The basestation will now check for available drivers.
- Please choose the first driver that is shown.

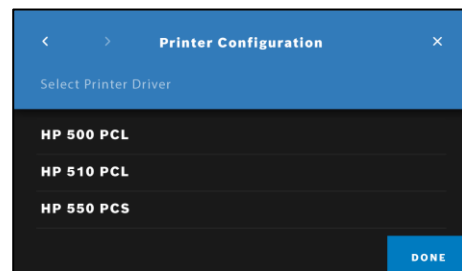

Driver name is an example

## 4. Confirm printer configuration

- After this, you will see a summary of the printer configuration.
- Press “SAVE”.

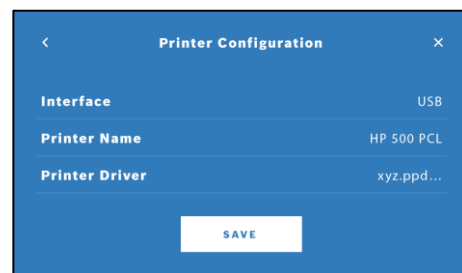

Once you have successfully configured a printer, reports can be printed.

## Option B: USB Printing


### INFORMATION

#### Precondition for USB Printing:

Your basestation is connected to a printer using a USB cable (usually provided by the printer manufacturer). The Vivatmo *pro* is designed to connect to the majority of common HP printers. (For a list of supported devices tested by BHCS, please contact BHCS Customer Support.)


## 1. Select Interface

- Select USB.

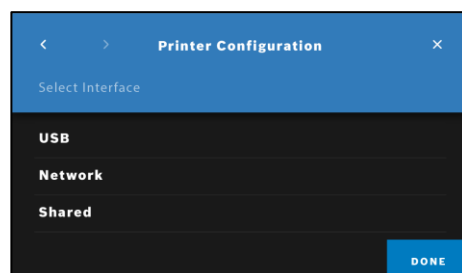

## 2. Select Printer


- The basestation will now search for available printers.
- Choose your printer and press “DONE”.

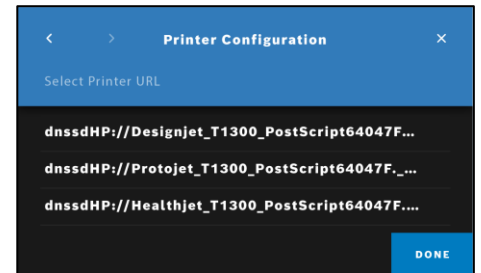

Printer name is an example

## 3. Select Driver


- The basestation will now check for available drivers.
- Please choose the first driver that is shown.
- Press “DONE”.

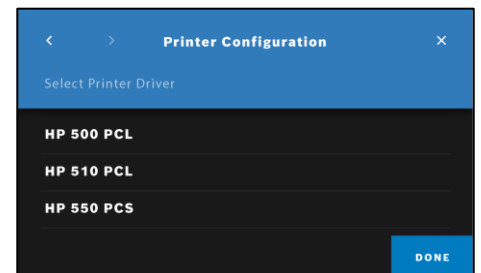

Driver name is an example

## 4. Confirm printer configuration

- After this you will see a summary of the printer configuration.
- Press “SAVE”.

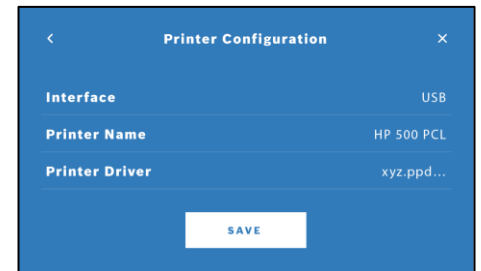

Once you have successfully configured a printer, reports can be printed.

## Printing via Ethernet cable connection

### INFORMATION


If you have a “non-HP printer” or the USB connection does not work, you can connect your printer to the basestation with an Ethernet cable.

### 1. Connect your basestation and printer with an ethernet cable.



### 2. Get your default printer IP address

- If your printer has a display, you can look up the IP address via the printer screen.
- If your printer does not have a display, you can print a configuration page with the information by pressing a printer-specific button.
  - In both cases, please check the Printer Manual for the precise procedure.

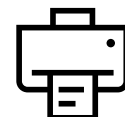

**3. The common IP format for HP printers is: 169.254.xxx.xxx**

**4. Define your Vivatmo IP address manually**

- Take your Printer IP and add “1” to the last digits  
Example:  
If your Printer IP is: 169.254.123.111  
Add “1” to “111” -> “112”  
Your Vivatmo pro IP address then is: 169.254.123.112

Printer IP: \_\_\_\_:\_\_\_\_:\_\_\_\_:\_\_\_\_

Vivatmo pro IP: \_\_\_\_:\_\_\_\_:\_\_\_\_:\_\_\_\_



Gateway IP: \_\_\_\_:\_\_\_\_:\_\_\_\_:1

**5. Define your Gateway IP address:**

- Use the first three number sets of the IP address and set the fourth block to “1”.  
In the example your Gateway IP then is: 169.254.123.1

**6. Now the network connection of your basestation has to be configured**

- On the **Network** Screen, select **Configure Network**.
- Select the **Network type Ethernet**.
- Disable **DHCP**.
- Enter the **Vivatmo IP address**.
- and the **Subnet Mask** (use 255.255.0.0).
- Enter the **Gateway IP** from above.
- Select the **Right arrow** or confirm with **DONE** to leave the screen.
- Confirm with **SAVE**.

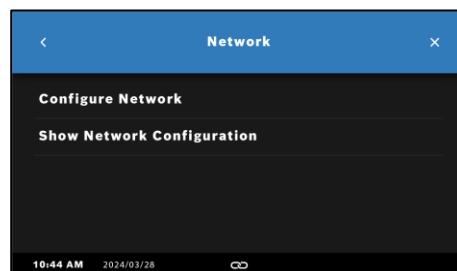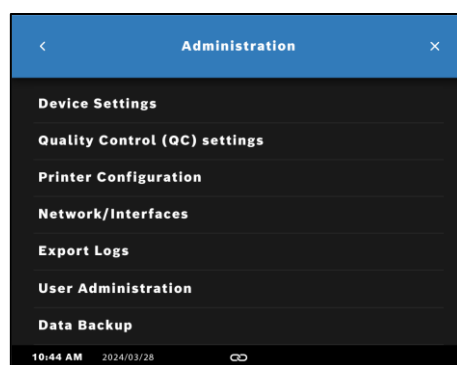

**7. After configuring your network, you can now continue with the configuration of the printer as described in Option A “Network and Shared Printing”.**

Once you have successfully configured a printer, reports can be printed.

If the the configuration process is unsuccessful or you are having trouble with a specific printer, please contact Bosch Healthcare Solutions Customer Support.

**Publishing details:**

Bosch Healthcare Solutions GmbH  
Stuttgarter Str. 130  
71332 Waiblingen, Germany

**In the U.S. Vivatmo pro is distributed by:**

**Methapharm, Inc.**  
11772 West Sample Road, Suite 101  
Coral Springs, FL 33065 USA  
Email: vivatmosupport@methapharm.com  
Phone: +1-833-887-7686