

# Vivatmo pro


## Initial setup of a new Vivatmo pro – How to

### First steps with your Vivatmo pro

The setup of the Vivatmo pro system includes the setup up of the handheld and the basestation. Please note that this document does not replace the instructions for use. Therefore, please take time to carefully read the instructions for use before using the Vivatmo pro.


Please also inspect the basestation, the handheld, the disposable mouthpiece and its pouch before usage. Do not use any defective or accessories to avoid malfunction or hazardous situations and contact your local distributor or our customer service team.

### Setting up the handheld

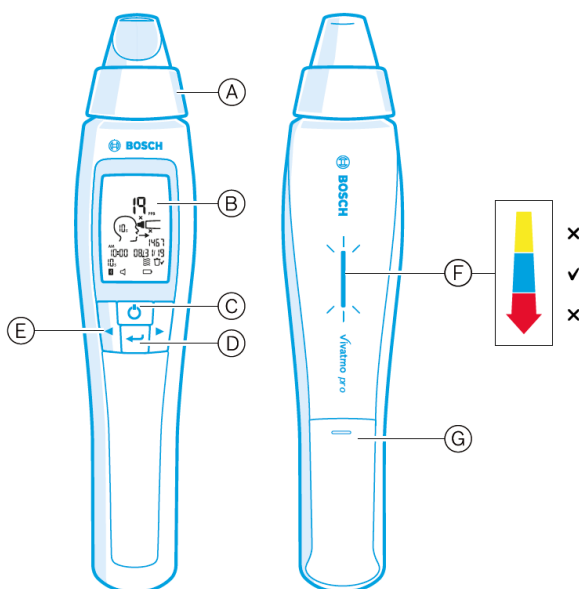

- (A) Disposable mouthpiece: Interface to breath into the Vivatmo pro handheld.
- (B) Display: Shows current and stored measuring data and device messages.
- (C) **ON/OFF** button: Activate and deactivate the device.
- (D) **ENTER** button: Press this button to start the regeneration or to confirm selection.
- (E) **ARROW** buttons: Press these buttons to change the settings and to access stored measuring data.
- (F) Breath intensity display:  
Feedback during the measurement  
**Yellow:** Caution, the breath intensity is too weak.  
**Blue:** The breath intensity is right.  
**Red:** Warning, the breath intensity is too strong.  
Procedure correct: Blue blinking.  
Procedure failed: Red blinking.  
When the handheld is placed on the basestation blue blinking indicates the charging of the battery.
- (G) Battery compartment.

1. Open the battery compartment on the back of the handheld.
2. Pick the handling tab with two fingers and insert the rechargeable battery into the battery compartment. Mind the triangle marks on the battery and inside the battery compartment.
3. Apply the battery cover.
4. Remove protective cap and place the handheld on the basestation cradle for charging.  
Charge before first use. The recharging time for a completely exhausted battery is about 12 hours. As long as the LED is blinking blue, the handheld is charging.
5. Press the **ON/OFF** button to turn on the handheld. You see all screen icons and the system performs a brief self-test. Then the software version on the handheld is shown.
6. Turn off the device before you replace the battery.

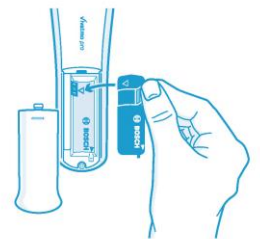


## Setting up the basestation

Make sure the basestation is connected to a suitable power supply. If necessary, please use a suitable adapter for your country. The Vivatmo *pro* basestation starts automatically.

When you start your Vivatmo *pro* basestation for the first time, you will be guided through a setup procedure of **5 simple steps**, which helps you to enter all the required settings.

### 1. Region and Language


- Select a region.
- The region selection defines the functionality approved for the region.
- The language screen is displayed.

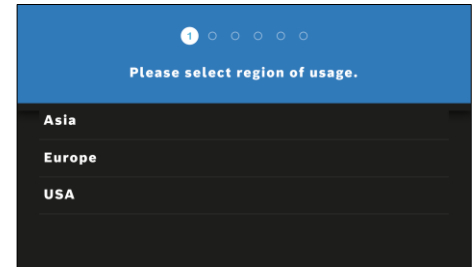

### 2. Select a language


- The **Date and Time** screen opens.

Press the **CONTINUE** button to finalize the remaining configuration steps of your Vivatmo *pro*.

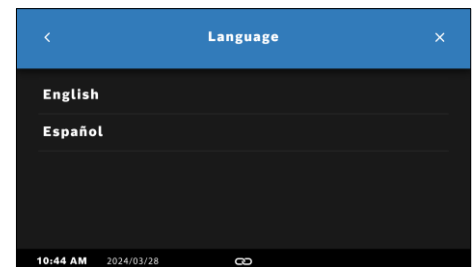


### 3. Date and Time

- Set up **date** and **time**. For details, see “Setting date and time<sup>1</sup>”.
- Confirm with **DONE** after successful setting.
- The **Administrator Account** screen opens.

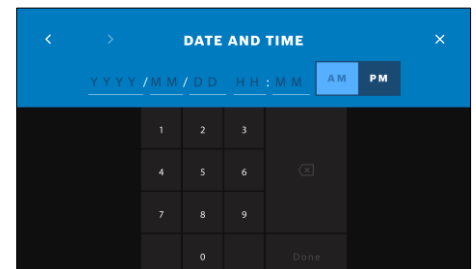

### 4. Administrator Account

- The **Administrator Account** has full access rights to the entire device.
- Create an administrator account by entering a username and an 8-digit passcode.
- Repeat the 8-digit passcode.
- Please note that only administrators are allowed to add new users<sup>2</sup> or to reset user passcodes.

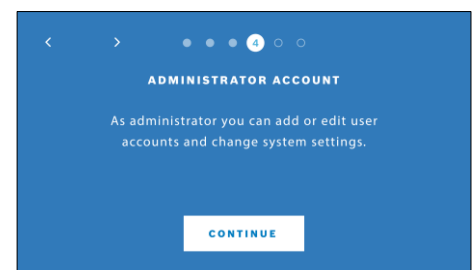

Select the **CONTINUE** button after the successful addition of an administrator. The **User Administration** screen opens.

<sup>1</sup> For details, see “Setting date and time in 6.5.1.” in instruction for use.

<sup>2</sup> For details, see “3.2.1 User concept” and “Add new user” in instruction for use.






## 5. User Administration

- In addition to the administrator account, other users can be created. These users are called “professional accounts” and have only limited access to the device<sup>1</sup>.
- Select **ENABLE** to activate the user administration.
- Create a new user by entering a user name and a 4-digit passcode.
- Repeat the 4-digit-passcode.
- Select **CONTINUE** after successful addition of a user and **see the list of user accounts**.
- Select the **Arrow** to leave the list.

On the next screen, select **CONTINUE** button.  
The **Handheld** screen opens.

## 6. Assign handheld

- To assign the handheld press **SCAN FOR HANDHELDS**.<sup>2</sup>
- Turn on your handheld.
- The basestation will start searching for the handheld via a Bluetooth connection.
- Wait until the scan procedure is completed. You see the list of active handhelds. If no handheld is active, you see the message: “No handhelds available”.
- Compare the number of the Vivatmo pro handheld in the list with the serial number beneath the symbol on the handle of the handheld.
- From the list, select the Vivatmo pro handheld which shall be installed with your Vivatmo pro basestation.
- The configuring device screen opens. The handheld and the basestation will now connect automatically<sup>3</sup>.
- The Vivatmo pro handheld and the Vivatmo pro basestation show the same date and time now.

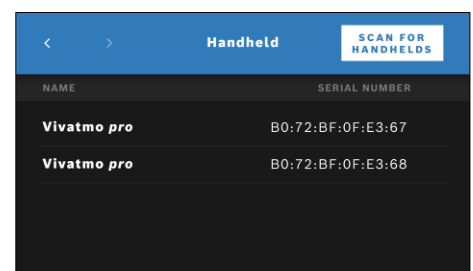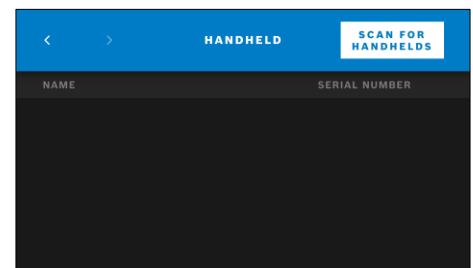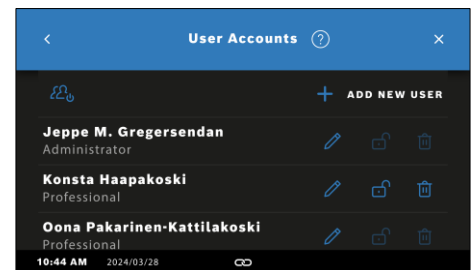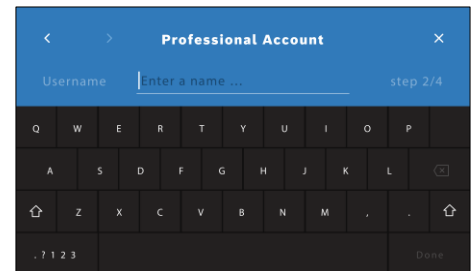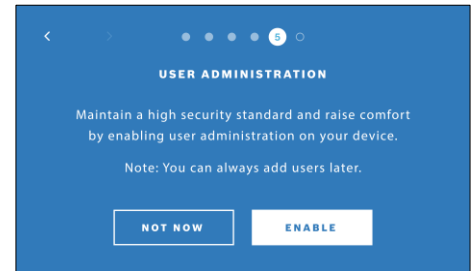

After successful assignment of the handheld, the **CONGRATULATIONS** screen opens up. Select the **FINISH** button. The setup procedure is **completed**.

If the setup process is unsuccessful, contact your local distributor.

<sup>1</sup> For details, see “3.2.1 User concept” and “Add new User” in instruction for use.

<sup>2</sup> For details, see “6.5.4 Assigning the handheld” in instruction use.

<sup>3</sup> If no handheld is active, the message: “No handhelds available” will show up. In this case, please try again or contact Customer Service.

### Publishing details

Bosch Healthcare Solutions GmbH  
Stuttgarter Str. 130  
71332 Waiblingen, Germany

### In the U.S. Vivatmo pro is distributed by:

Methapharm inc.  
11772 West Sample Road, Suite 101  
Coral Springs, FL 33065 USA  
Email: vivatmosupport@methapharm.com  
Phone: 1-833-887-7686